# Bivalve Orthosis Donning Instructions & Spinal Orthosis Protocol

- Before donning orthosis patient should be sure to wear a tight-fitting, cotton t-shirt or stockinette. Wrinkles under the jacket can cause irritation and skin breakdown.
- The orthosis can be cleaned with soap and warm water or rubbing alcohol.
- Follow physician and orthotist instructions for length of orthosis wear.
- Follow physician and orthotist instructions for activities acceptable while wearing orthosis.
- Physician's orders should supersede all protocol.


**Spinal**  
Technology, Inc.

## Orthosis Donning Instructions

### Step 1

Patient should be on back with hips flexed slightly. Palpate waist (area above hips and below ribs) to demonstrate proper position of orthosis location.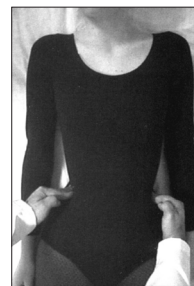

### Step 2

Raise patient's arm above their head. Log roll patient toward raised arm onto their side. Be sure hips and upper body move together without twisting.


### Step 3

Identify top & bottom of orthosis. Place back section of orthosis behind patient. Waist rolls in orthosis should align with patient's waist.


### Step 4

Roll patient onto back with orthosis under them. Recheck alignment of waist rolls with patient in this position. Move orthosis accordingly.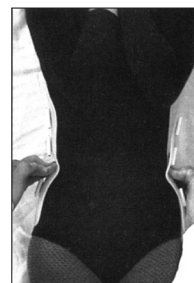

### Step 5

Place front section of orthosis on patient noting top and bottom. Straps and loops should be across from each other.

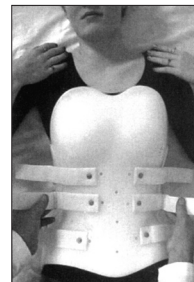### Step 6

Fasten middle straps first pulling both sides evenly. Tighten top or bottom straps next in same manner. Tighten all straps as needed.

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